



## Control Handle Replacement Instructions • Zinger Chair USA

1. Remove the battery, set both parking brakes, and rotate the Zinger onto its back.

2. Using a phillips screw driver, remove the three smaller screws on the underside of the control handle, and lift the bottom handle section off.

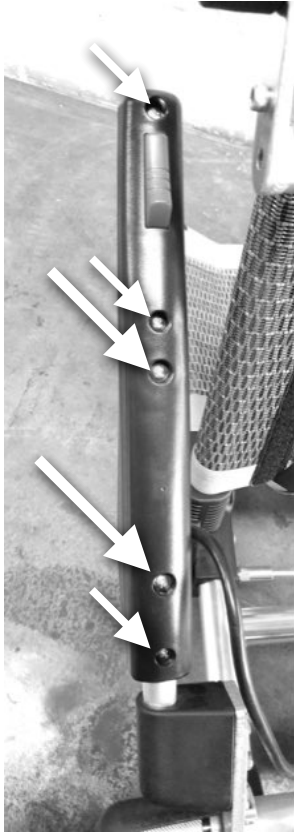

3. Remove the screw **D** to separate the control handle from the metal lever arm.

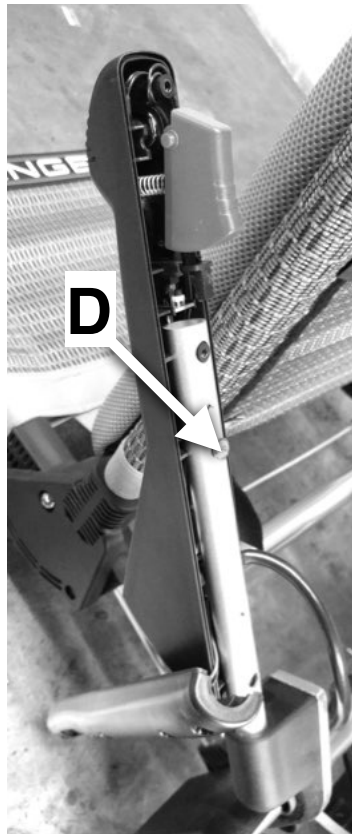

4. By gripping the outside ends of the white connector, pull the connector free from its socket.


5. Install the white connector into the new control handle then install screw **D** through the metal lever arm.



6. Check that the trigger's spring is on the finger molded into the inside of the handle and that the trigger's tab is in the sensor's fork as seen at arrow **A**. The trigger hinge pins **B** should be directly above the hinge cradle **C** as seen at left.

7. Close the handle sections together, taking care that no wires are pinched between the two halves. When the two halves are comfortably fitted together install the two blunt end screws first (large arrows in first photo). Then install the three sharp end screws.

8. Set the chair on its wheels. Leaving the parking brakes on, install the battery. Turn the power on if its not already, and try the trigger for proper function.

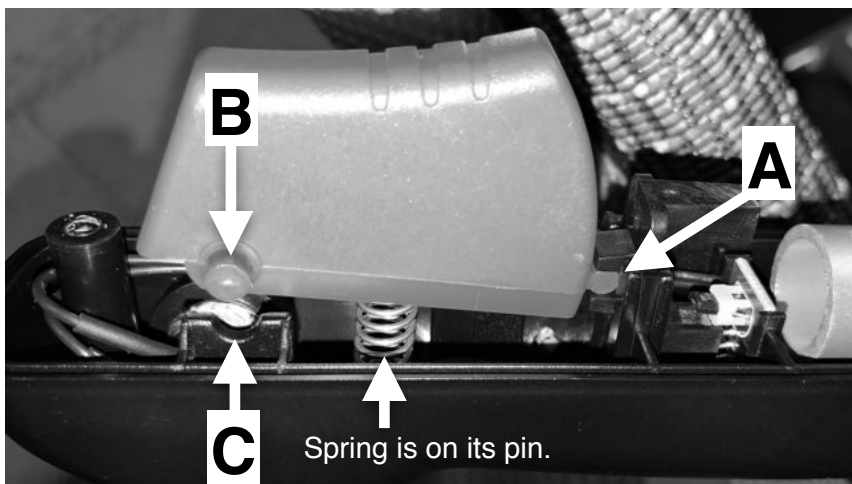

Please call if you have any difficulties.  
800-958-8324