


Oxygen Bag Installation Instructions Zinger Chair

1. As tightly as possible, wrap both pairs of Velcro straps, 1 and 2, snugly around the curved seat back bar. It does not matter which section is on top of the other, the Velcro works either way. Squeeze the straps against the bar in order to make the Velcro secure.

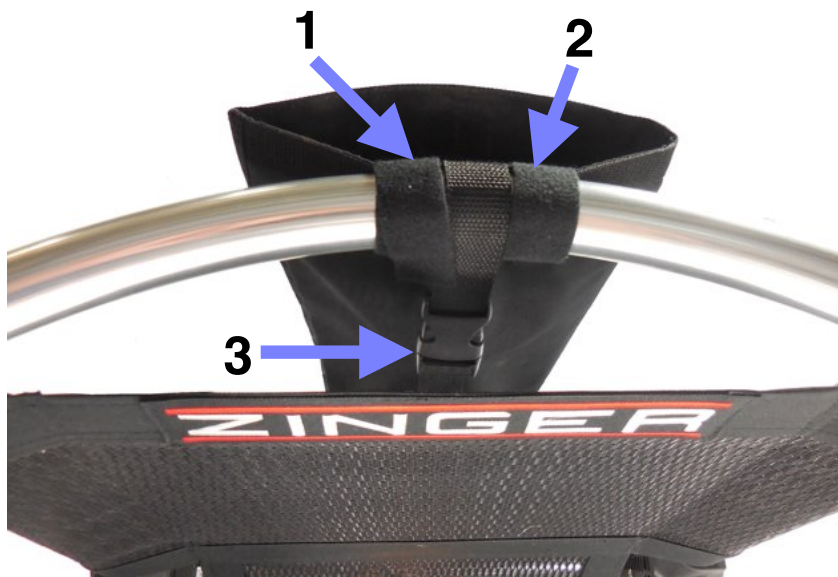

2. Connect the buckle 3, and pull the lower tail until there is no slack in the belt.

3. Open the two lower velcro straps. Avoiding any twist, wrap them under the lower cross tube 1 and around the front of the vertical tube 2, so that the tails wrap vertical tube 2 front to back (see blue line) and so that the velcro surfaces mate. Take care to center the bag on the chair.



Hint: To get your O2 bottle into the bag, open the Zinger and rotate it onto its back, lay the bottle on the floor, and slide it into the bag.

If you have questions please call us at 800-958-8324