

ZINGER ARM REST INSTALLATION INSTRUCTIONS

1. In the clear plastic fastener box that came with your Arm Rests, you will find two 5mm short hex wrenches. Using the two hex wrenches, hold the hex-head bolt at location **A** stationary while removing the outer hex-head screw opposite it, at location **B**.

2. Stack one of the four new screws **10** with a small flat washer **11**.

3. Using one of the 5mm hex wrenches, hold the hex-head bolt at location **A** stationary. While using the longer 4mm hex wrench, thread the new screw **10**, with washers installed, into location **B** and tighten firmly.

4. Using the same process as in step 1, remove the outer screw at location **D**.

5. Stack another of the new screws **10** with: a spring washer **12** followed by one of the large flat washers **6**. Pass screw **10** through the hole in the Arm Rail Bracket **C**.

6. Install spacer **8** over the screw with its smaller diameter side facing the Arm Rail Bracket **C**.

7. With one hand, squeeze the two black plastic hinge parts together. With your other hand, use the 4mm hex wrench to begin threading screw **10** into the hole at location **D**. Continue threading for at least three full revolutions.

*Be very careful when beginning the threading of screw **10**, that it starts easily and correctly so as not to cross-thread thus ruining the screw.*

8. Finally, using one of the 5mm wrenches, hold the hex-head bolt at location **E** stationary while using the 4mm hex wrench to firmly tighten screw **10**.

Follow the same procedure to install the right side Arm Rail.

If you have any questions please call us at 800-958-8324

